

Lebih sensitif dari deteksi Antibodi **BIOCREDIT COVID-19 Ag**

SARS-CoV 2 Antigen test

- » The first commercialization Black Gold Particle technology In the world (high sensitivity and specificity)
- » User friendly Dual Color System (Control line: Red, Test line: Black)
- » Rapid Test Time (5-8 Min)
- » Simple detection of SARS-CoV2 antigen

MAY 9, 2020 - The FDA has issued the first emergency use authorization for a COVID-19 antigen test, which is a new type of test that detects virus proteins. It is a diagnostic test that indicates whether someone has the coronavirus at the moment. Samples are taken with a nasal swab.

The new antigen test is a diagnostic tool like the PCR test, but instead of looking for the genetic sequence of the coronavirus, it looks for antigens, or molecules on the surface of the virus.

Core Technology Black Gold Particle




Black Gold Particle

Differentiated Black Gold Conjugate Technology



Dual Color System

8 minutes is ok!

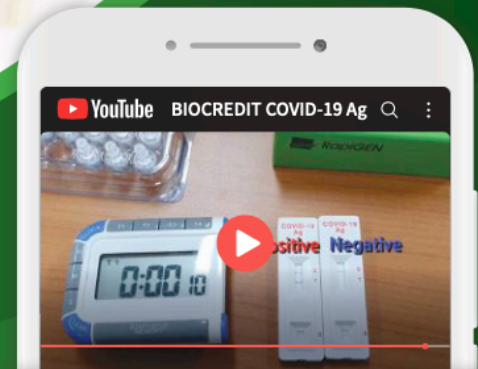

YouTube **BIOCREDIT COVID-19**

Rp. 6.930.000 / kit

BIOCREDIT COVID-19 Ag rapid test kit

Intended Use	Detection of SARS-CoV2 antigen 20
Package	Tests / kit
Storage	1-40°C
Specimen Type	Nasopharynx / Nasopharyngeal
Shelf life	12 months from manufacture
Time to result	date 5-8 minute



COVID-19 Rapid Diagnostic Test

PROCEDURE

Insert the swab specimen and swirl the swab 5-10 times.

Remove the swab while gently squeezing the head of the swab.


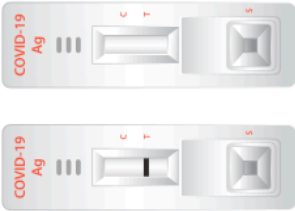
Close the assay diluent tube with a filter cap securely.

Invert the assay diluent tube and gently squeeze it to draw 3-4 drops (90-150µl) into a sample well on the device.

Read the result within 5-8 minutes.



INTERPRETATION

NEGATIVE	POSITIVE	INVALID
		
<p>One red line "C" within the result window.</p>	<p>Two bands, black To test line and red "C" control line within the result window.</p>	<p>No "C" line within the result window. It is recommended that the specimen be retested.</p>

Distributed by



"Your research and diagnostics partner"

PT. MEDIPATH BIOSAINS INDONESIA

Komplek The Beige, Blok B1.
Jl. Raya Cilangkap, Cilangkap - Cipayang.
Jakarta Timur 13870
Indonesia

Phone : (62) 21 8430 4909
Fax : (62) 21 8430 0198
Email : info@medipathbiosains.co.id

BioVendor – Laboratorni medicina a.s.
Karasek 1767/1, 621 00 Brno, Czech Republic
Phone: +420 549 124 185 · Fax: +420 549 211 460
E-mail: info@biovendor.com

› www.biovendor.com

